

# Ziza™ Seat - Tray

Art. No. SC605



## UK User Manual

### Mounting the tray

Using the tray requires that the Brace is also fitted. Then the tray is easily mounted by simply sliding it alongside the Brace and pushing until it clicks on - see fig. a and b. Removing the tray is done by dragging outwards on both sides where the tray is clicked on (fig. c) and then pulling towards the front - see fig. d.

## DK Brugervejledning

### Montering af bakke

Brug af bakken forudsætter, at bøjlen er monteret. Derefter er bakken nemt monteret ved at lade den glide langs bøjlen og presse, indtil den klikker på plads - se fig. a og b. Bakken fjernes igen ved at skubbe udad i begge sider i de to punkter, hvor bakken blev klikket på (se fig. c), og derefter trække mod fronten - se fig. d.

## SE Användarmanual

### Montering av bordet

För använda bordet krävs att staget också är monterat. Sedan monteras bordet enkelt genom att helt enkelt skjuta det längs staget och trycka tills det klickar på - se fig. a och b. Att ta bort bricken görs genom att dra utåt på båda sidor där bordet klickas på (fig. c) och sedan dra framåt - se fig. d.

## NO Brukerveiledning

### Montering av brettet

Bruk av brettet krever at braketten også er montert. Deretter monteres brettet enkelt ved å skyve det langs braketten og skyve til det klikker på plass - se fig. a og b. Å fjerne brettet gjøres ved å dra utover på begge sider der brettet er klipset fast (fig. c) og deretter trekke mot forsiden - se fig. d.

## ZH 使用说明书


### 安装餐盘

使用餐盘时需要先安装好前围，餐盘沿着前围顶部滑动，直至其卡合上，见图a和b。  
拆卸餐盘时候，将餐盘两侧向外拉，脱离卡合的位置（图c），然后向前拉餐盘并将其取下，见图d。

## RUS Руководство пользователя

### Установка подноса

Использование подноса требует предварительной установки фиксатора. После этого для установки подноса достаточно просто продвигать его вдоль фиксатора, нажимая на него до щелчка - см. рис. а и b. Чтобы снять поднос, необходимо потянуть его в направлении наружу в местах боковых защелок (рис. c), после чего потянуть его по направлению вперед - см. рис. d.

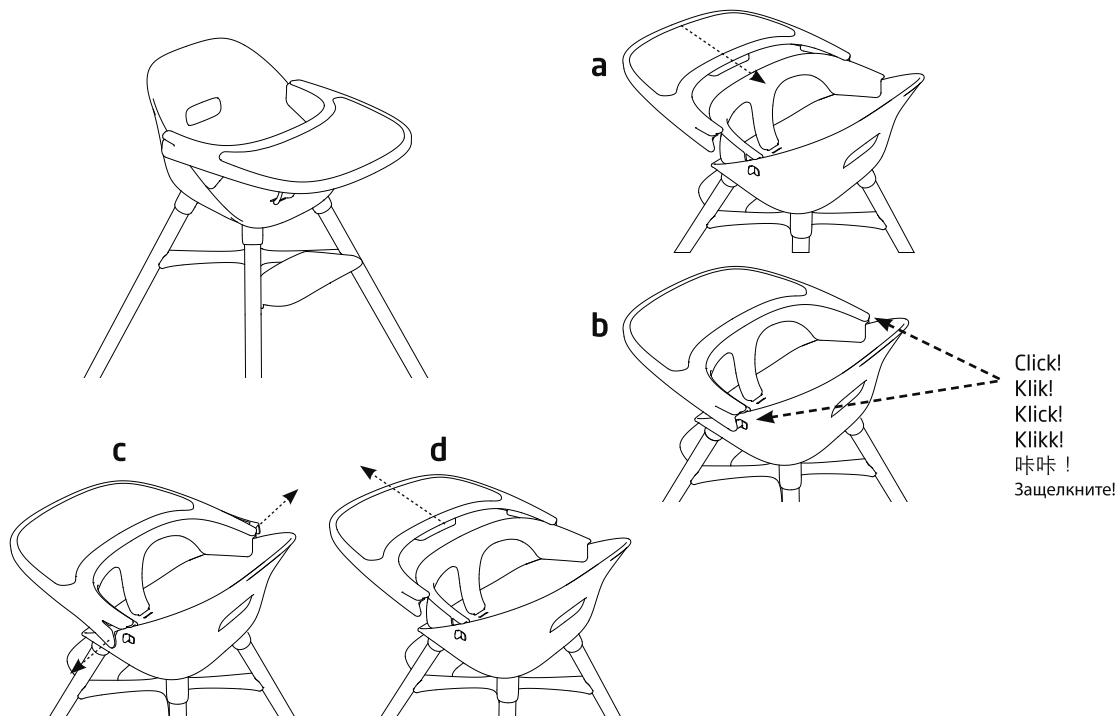

Ziza™ is a high quality brand developed and manufactured by Safe and Care Co.

Europe: Marketing and Development | Safe and Care Co. ApS | Kirkegade 4 | 8881 Thorsoe | Denmark | [contact@safeandcareco.com](mailto:contact@safeandcareco.com)  
Asia: Manufacturing | Safe and Care Co. | Jiangbei Road, Xiani Village | 523655 Qingxi Town | Dongguan City | Guangdong Province | China

[www.safeandcareco.com](http://www.safeandcareco.com)